

Medit Link v2.0.0 - Release Note

Performance

- There has been some latency time when logging-in and using Medit Link Application. This problem has been solved.
- Previously, Medit Link entered to the offline mode too often even when just viewing the data. We minimized the case for entering offline mode.

Case Box

- The name of 'Scan Box' has been changed to 'Case Box'.
- 'New Patient' button is replaced by 'New Case' button. You can make new case by simply entering or selecting the patient name. If there is no patient whose name matches with the name you entered, new patient information will be created automatically. You can manage patient information in the patient list which pops up when pressing the list icon in 'New Case Information' dialog.
The 'Case Name' field is added for you to manage the cases easily. The case name is filled automatically when entering the patient name, and you can freely set or modify the case name.

Modify Case Information

Patient Name* MeditLink_Test ✓ + ✎ ☰

'MeditLink_Test Male 2000-04-15' is selected.

Case Name* MeditLink_Test's Case

ⓘ Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties on order.

[Terms & Conditions](#) Cancel OK

Patient List



Select a patient in the list to set the patient of the case. You can add or manage the patient information in Medit Link.

Search By Patient Information +


Patient Name	Gender	Birth Date	Patient ID	Registered Date	Number of Cases	Recent Visit Date	
Jake Lee	Male	2000-12-12	00000152	2019-01-04	1	2019-01-04 오전 9:48	✎
Micheal Lee	Male			2019-01-03	1	2019-01-03 오후 5:10	✎
Crown				2019-01-03	1	2019-01-03 오후 3:58	✎
Initial Test				2018-12-28	1	2018-12-28 오후 2:42	✎
MeditLink_Test	Male	2000-04-15	0000001	2018-12-28	1	2018-12-28 오전 11:30	✎
Scanned Date				2018-12-26	1	2018-12-27 오후 6:06	✎
Recovered Data (500019)	Male	1970-01-01		2018-12-27	1	2018-12-27 오후 6:06	✎

Close Select

You can modify the case name and the relation between case and patient by pressing the button in 'Case Detail' page.

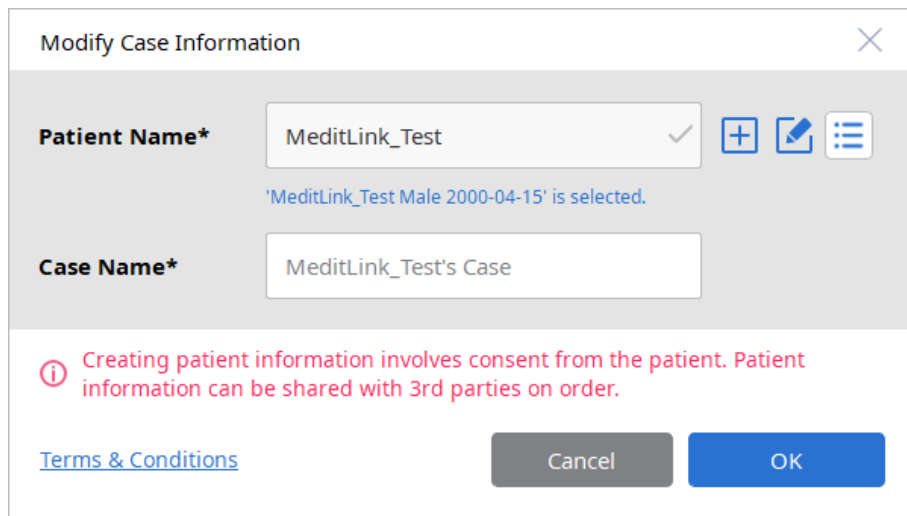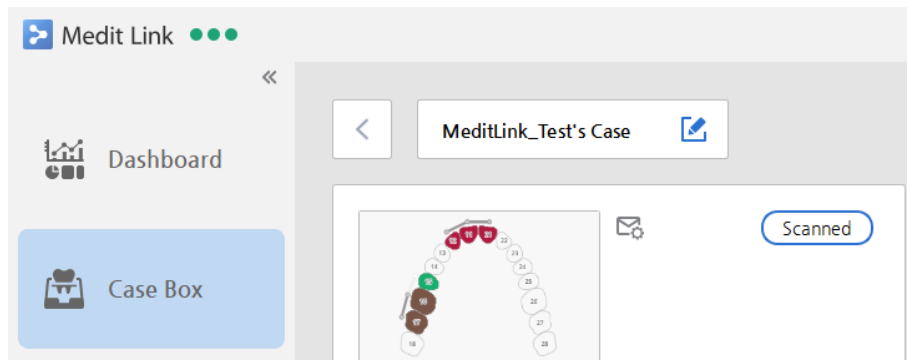


- The new 'Chart View' is added to the 'Case Box', which helps you to identify cases more easily.


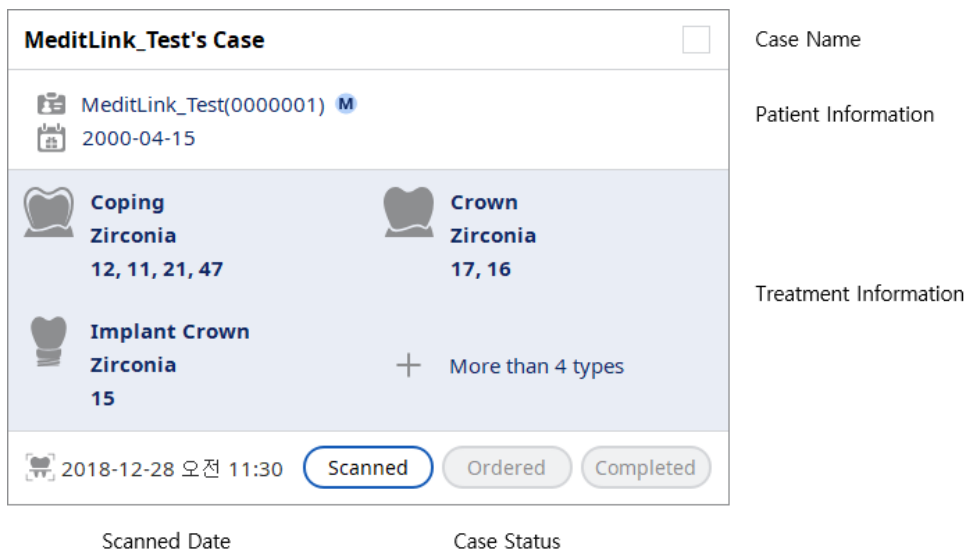


- The 'Scanned Date' field is added, which helps you to find cases based on the date when the patient recently visited.


- The 'Clone' button is added to the 'Case Detail' page. You can make the copy of the case easily by pressing this button. Previous confusing UI for cloning, which pops up when pressing 'Scan' button, has been removed.

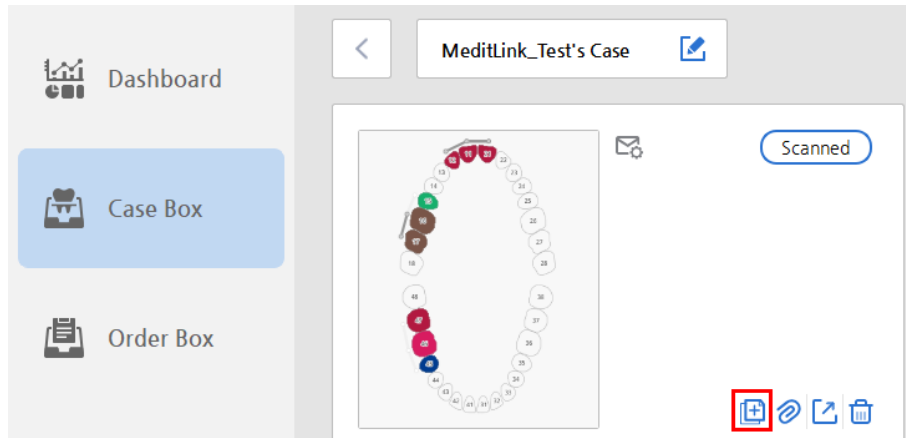

- The 'Share' button is added to the 'Case Detail' page. You can share the scan data with others by pressing this button. It generates public link which displays the data in web 3D viewer. For the time being, please use the 'Chrome' browser when accessing the web 3D viewer with the public link. Other browsers may work incorrectly with our web 3D viewer.

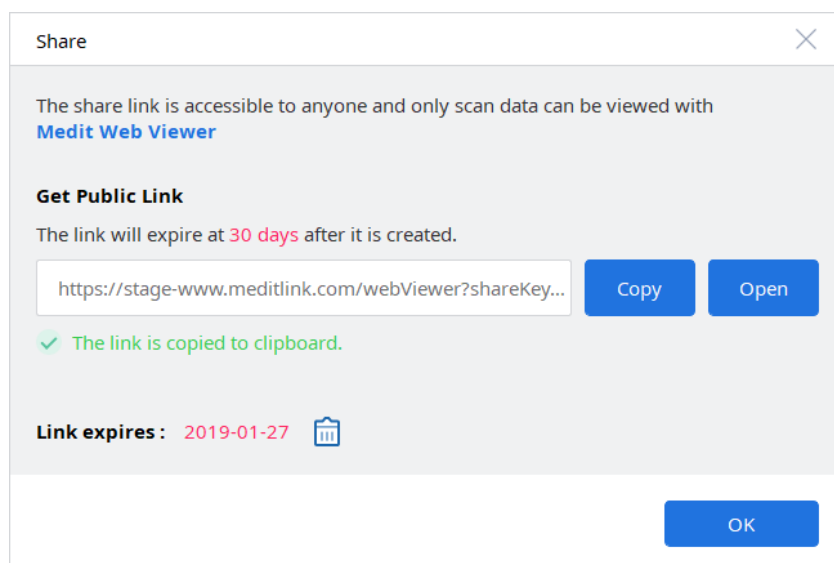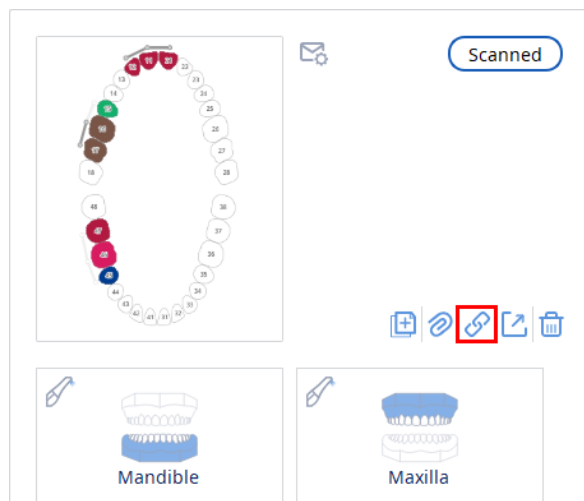

- The progress indicator, which show the progress of processing data, is added.

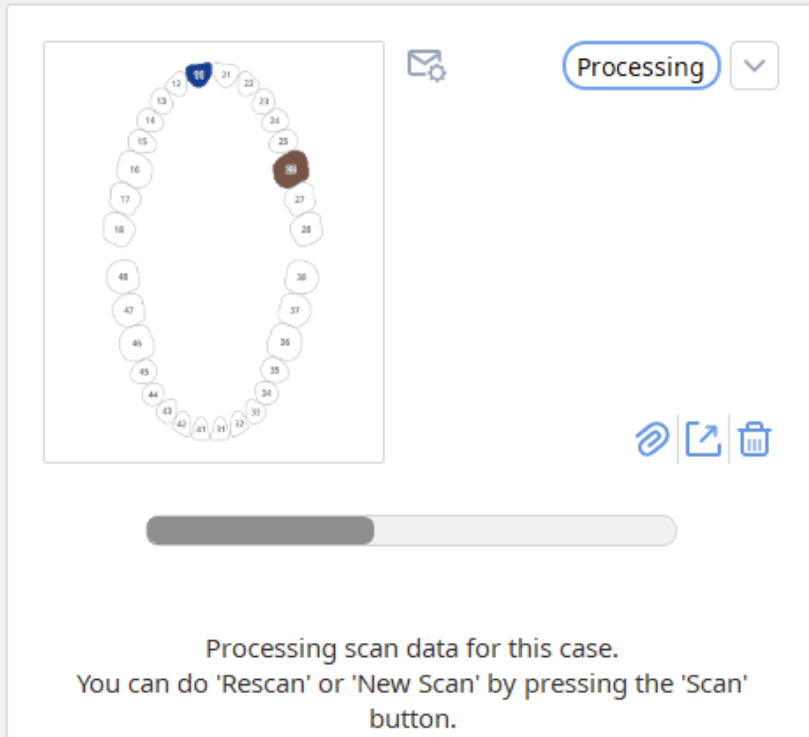


The screenshot displays a software interface for processing scan data. On the left, a circular diagram consists of 48 numbered nodes arranged in a ring. Node 15 is highlighted in blue, and node 26 is highlighted in brown. To the right of the diagram is a gear icon. Further right, a button labeled 'Processing' is shown with a dropdown arrow. Below these elements are three icons: a link icon, a share icon, and a trash icon. A horizontal progress bar is positioned below the icons, with a dark grey segment on the left indicating the current progress. At the bottom, a text message reads: 'Processing scan data for this case. You can do 'Rescan' or 'New Scan' by pressing the 'Scan' button.'

- The ordering process has been simplified. You can make an order of case regardless that the lab has price information for the product. The 'Share Patient Name' is checked on by default, and it remembers the status of it you used.

Order Case ✕

Partner* Go To Partner


Payment Details



No Price Information

Case Name* **Share Patient Name** ?


Patient Name*

Delivery Date* 

Memo

- Previously, you could not make a copy of a case when the case has been ordered. This problem has been fixed, and you can make a copy of an ordered case.

- The 'Pre-Operation Model' option is added, and you can set this option when defining the form information. If you set this option on, the iScan will show you additional stage for scanning. The 'Pre-Operation Model' option is for scanning the teeth before the preparation.

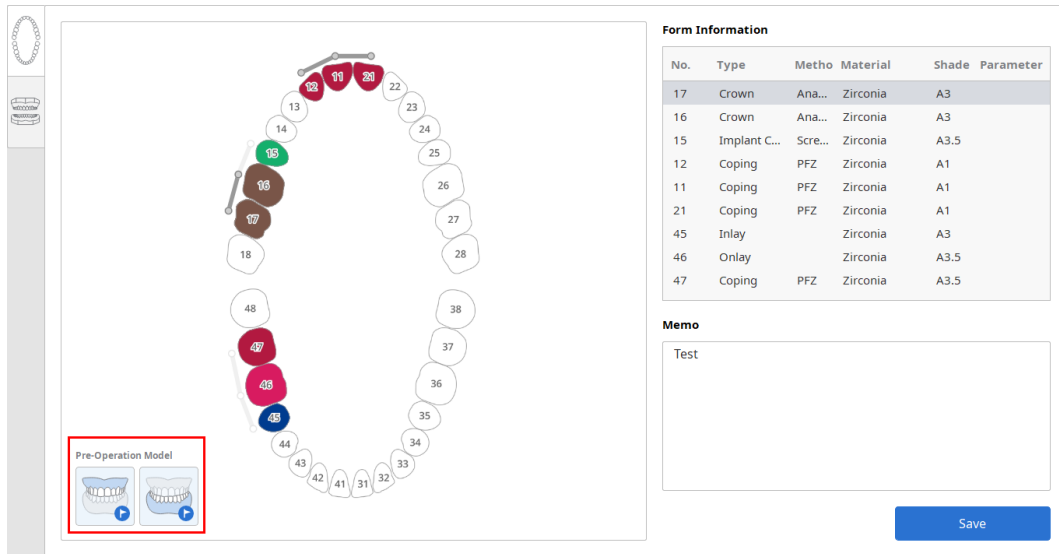

The screenshot shows a dental software interface. On the left, there is a vertical toolbar with icons for a tooth chart and a 'Pre-Operation Model' option, which is highlighted with a red box. The main area displays a tooth chart with numbered teeth (1-48) and a 'Pre-Operation Model' option highlighted in a red box. On the right, there is a 'Form Information' table and a 'Memo' field.

No.	Type	Metho	Material	Shade	Parameter
17	Crown	Ana...	Zirconia	A3	
16	Crown	Ana...	Zirconia	A3	
15	Implant C...	Scre...	Zirconia	A3.5	
12	Coping	PFZ	Zirconia	A1	
11	Coping	PFZ	Zirconia	A1	
21	Coping	PFZ	Zirconia	A1	
45	Inlay		Zirconia	A3	
46	Onlay		Zirconia	A3.5	
47	Coping	PFZ	Zirconia	A3.5	

Below the table is a 'Memo' field with the text 'Test' and a 'Save' button.


- The scan data for occlusion is also generated, and they are sent to the lab when ordering the case.

In Box

- Previously, you should press 'Share To Work Box' after confirming the order. This has been simplified by automatically making the case shared to the Work Box when pressing the 'Confirm Order' button.
- The default search criteria has been changed to 'Last 30 Days'.

Work Box

- The name of folder for 'Export for CAM' is set automatically by mixing clinic name and patient name.

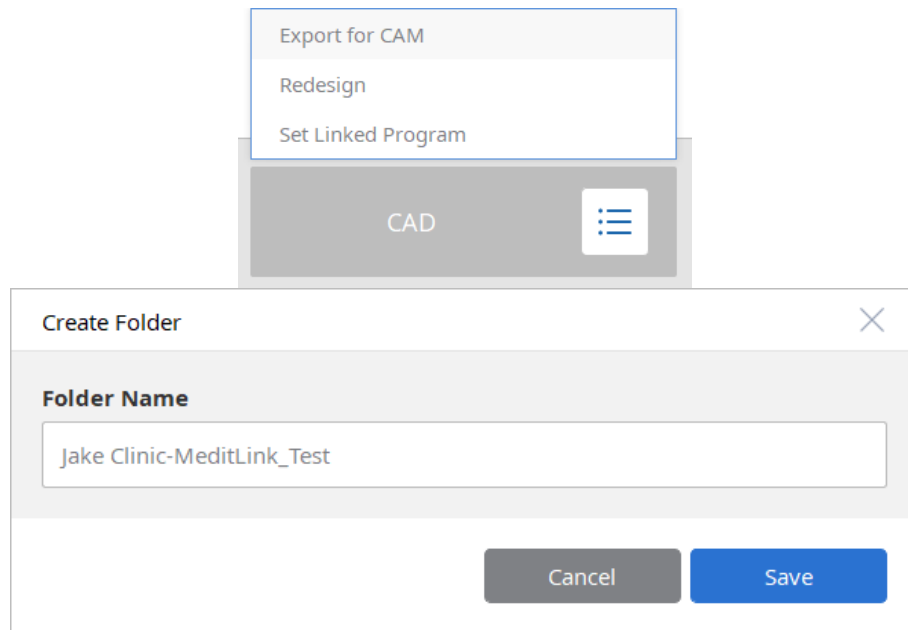


- The material configuration for defining dentalProject file has been added. There are 5 configurations such as Default, JawMotionImport, LAVA, Partial, VITA.
- You can choose dentalProject file when there are two or more dentalProject files.
- The behavior for defining dentalProject file is same with Exocad DentalDB 2.2 Valletta (Build 6654).

Web

- The process which should be done before making an order has been simplified by adding the concept of Partnership. You can make a partnership without the price information, and you can make an order or receive an order if you have partnership with someone.
- You can invite a partner in Partners menu. Once the partner accepts the invitation, the partnership with you will be made automatically.
- The UI for displaying organization information has been improved. You can check it by pressing 'Preview' button in 'Organization Information' tab in 'My Organization' menu.

Etc.

- You can check the password by pressing the button in the Password field.

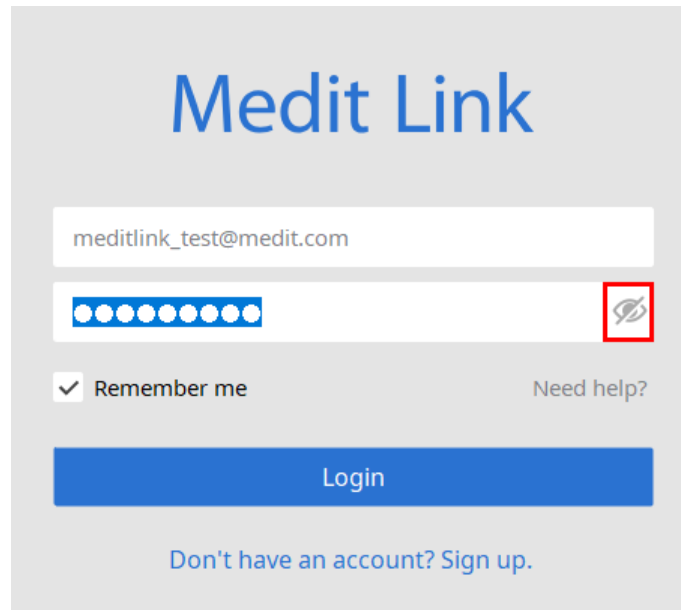

- You can collapse the left-side bar by pressing the button at the top. You can see wider view by collapsing left-side bar.

